


KARIES SCREEN TEST. DE

Check up is good -
testing even better!


Instructions for the KariesScreenTest buffer

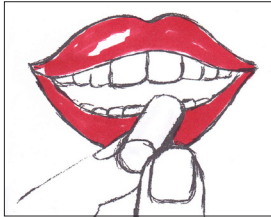


Chewing a paraffin capsule,
swallow saliva in the first minute.




Then collect the saliva for 5 minutes.

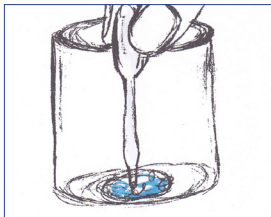


The secretion rate can be determined with the saliva sample.

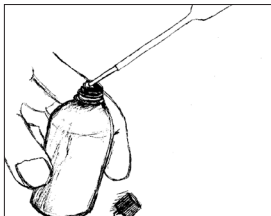

Then take 1ml of saliva and put it in the bottle with the indication
solution and shake.

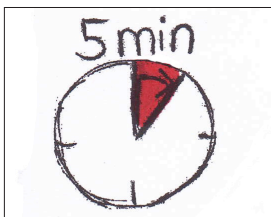

After 5 minutes determine the pH value by immersing the
measuring strip.

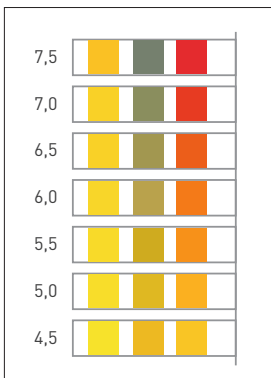

ph > 6,0 very good buffer capacity

ph > 5,0 good buffer capacity

ph < 5,0 poor buffer capacity

ph < 4,0 very poor buffer capacity